


InstaShiatsu+ Massage Chair MC-2000

Massage Head Distribution Valve Replacement


1. Gather the necessary tools: Phillips head screwdriver and wire cutters.
2. Locate the two screws that secure the bracket to the massage head.
3. Using the Phillips head screwdriver, carefully unscrew and remove both screws from the bracket.

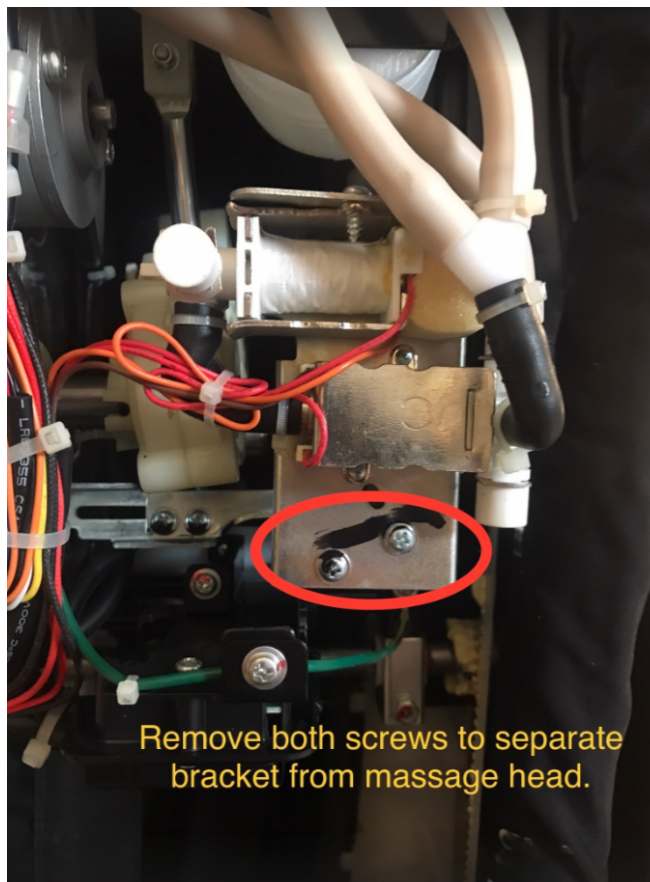


4. Once the screws are removed, gently separate the bracket from the massage head.



5. Identify the three zip ties that secure various components.

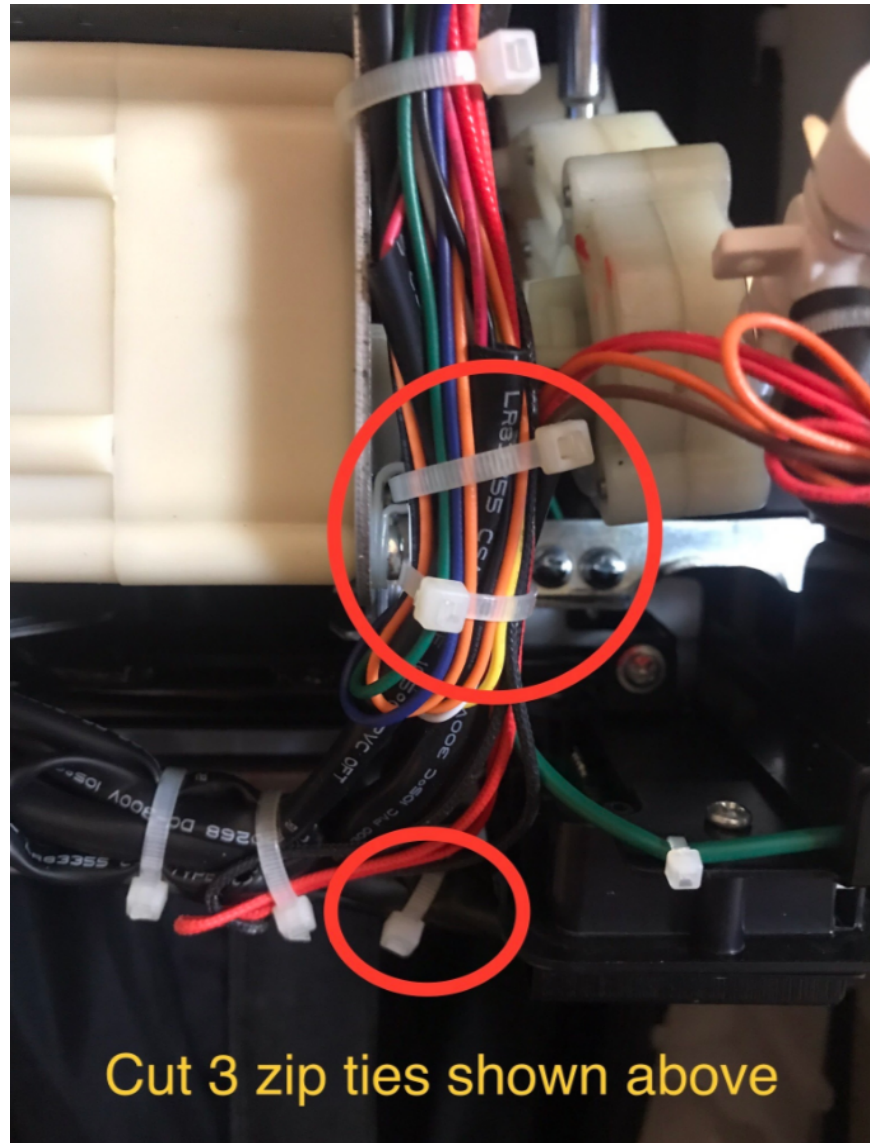

6. Using the wire cutters, carefully cut the three zip ties, one by one, to release the tension and remove them.
7. Locate the two screws that secure the circuit board in place.
8. Use the Phillips head screwdriver to unscrew and remove both screws from the circuit board.

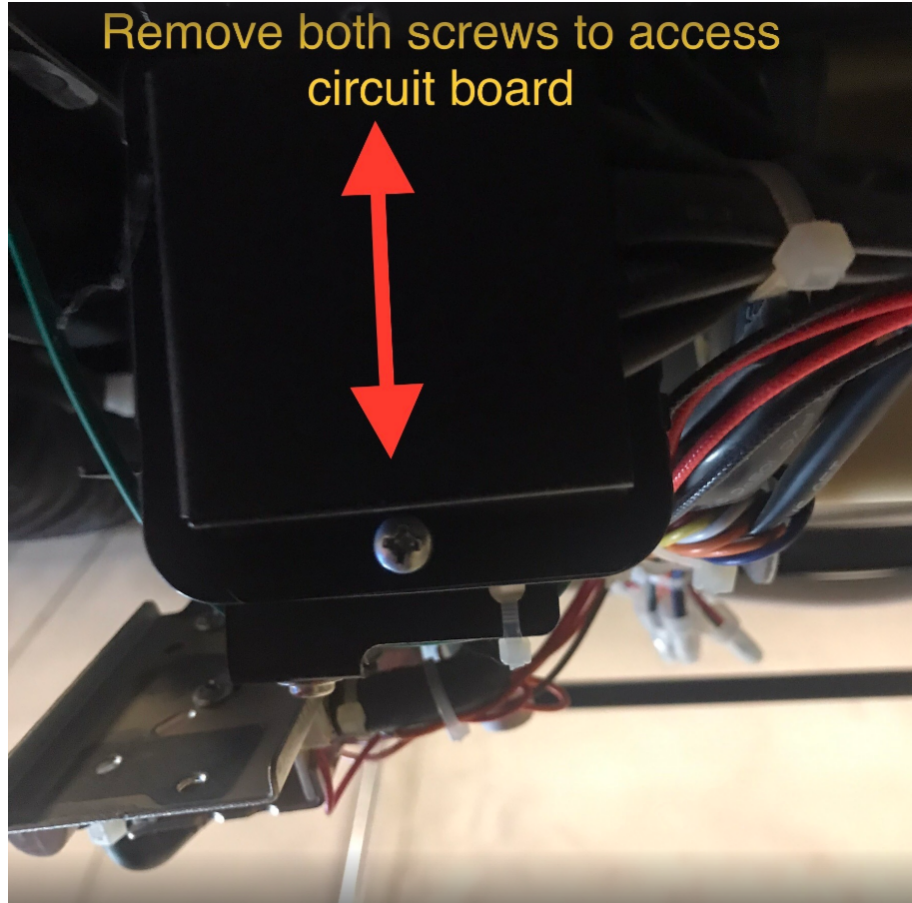

9. Once the screws are removed, carefully lift the circuit board to access the wiring harness.
10. Locate the wiring harness connected to the circuit board.
11. Gently disconnect the harness by pulling it apart. Avoid pulling on the wires directly to prevent damage.



12. The bracket, circuit board, and harness are now separated and ready for further inspection or repair.

Remember to exercise caution while performing these steps to avoid any damage to the components. If you're unsure about any part of the process, it's recommended to consult the product's user manual or seek assistance from a professional.



truMedic Canada, Inc.
2036 Ellesmere Road
Scarborough, Ontario M1H 2V6

888-264-1766 x203
www.trumedic.com
hr@trucooredistributors.com
payroll@trucooredistributors.com

To install the replacement massage head distribution valve, please follow the instructions provided in reverse order.